



DualTec™ Kit

Complete implantation system

The DualTec™ IOL Injector is a single use lens injector for the implantation of one- and three piece acrylic IOLs. The DualTec™ Kit features complete sets of a DualTec™ Injectors and Locking Wing System™ cartridges required to implant your IOL of choice.



Main Features & Benefits




- ✓ **2.2 mm incision** (*one piece acrylic IOLs*)
2.8 mm incision (*three piece acrylic IOLs*)
- ✓ **PUSH or TURN**
PUSH for one piece IOLs,
TURN for three piece IOLs
- ✓ **Single use**
Safe, hygienic, constant quality
- ✓ **Cartridge**
Convenient cartridge loading






DualTec™ Kit Loading Instructions


>> Load

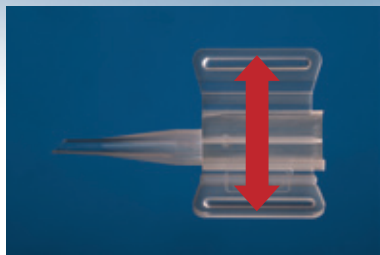

1. Open cartridge;

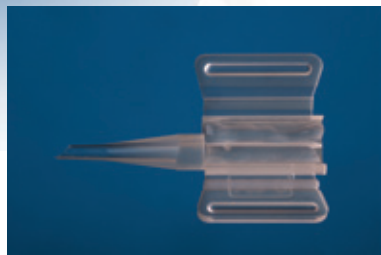


2. Fill cartridge with viscoelastic solution;

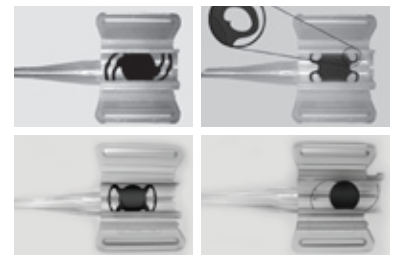

3. Push the lens with the haptics under the rim;




4. Close cartridge until a 'click' is heard;

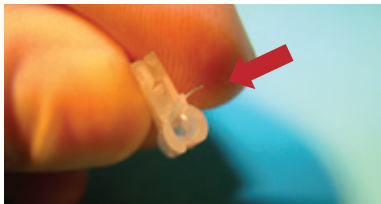


4a. For three-piece IOLs the rear haptic should stick out of the cartridge;

>> Place

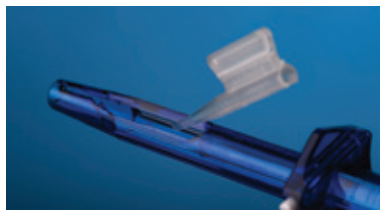

5. Place cartridge into DualTec™ injector;

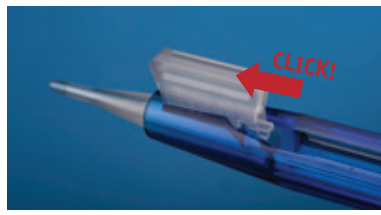

6. Move cartridge forward until a 'click' is heard;




7. Slowly guide the plunger towards the loading chamber of the cartridge;

>> Insert



8. Choose PUSH or TURN using the white switch:

PUSH for one piece IOLs,
TURN for three piece IOLs;

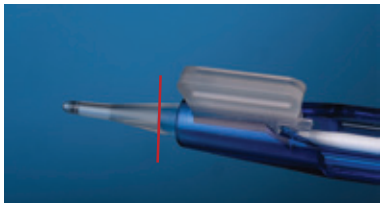

9. Move plunger forward into compression section. Retract plunger and move forward again into compression section;



10. Introduce tip of cartridge through the incision and move plunger forward until IOL is out of the cartridge.

Packaging

Injector + cartridge (in blister, box of 10)

- OD 655 DualTec™ Kit 3P Acrylic 2.8
- OD 665 DualTec™ Kit 1P Acrylic 2.0

Cartridge (in blister, box of 10)

- OD 502 LWS™ Cartridge 3P Acrylic 2.8
- OD 522 LWS™ Cartridge 1P Acrylic 2.0

